

# News letter


This letter is to inform our customers that da Vinci Xi endoscope can be sterilized using the ECO cycle on the RENO-S130 low temperature peroxide sterilizer.

An evaluation of device compatibility and a validation of sterilization efficacy were performed according to the procedures of Intuitive Surgical and in accordance to ISO14937:2009 sterilization of health care products –General requirements for characterization of a sterilizing agent and the development, validation and routine control of a sterilization process for medical devices. These studies serve to successfully validate the RENO-S130 using the ECO cycle for use in sterilizing all reusable Xi endoscope.

Please refer to the reprocessing instructions provided by Intuitive Surgical for a complete description of the cleaning process validated for da Vinci endoscopes.

## Sterilization Process Parameters

- ◆ Xi Endoscopes  
Stainless steel  
Sterilization Tray  
PN400498

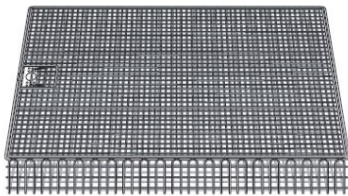

STERILIAZION MACHINE	CYCLE	Sterilization Wrap (MAX thickness)	LOAD CONFIGURATION
Renosem, RENO-S130	ECO	400 thickness	Single tray on top shelf

- ◆ Warning. Always use a 400 weight thickness sterilization wrap or lighter  
Use of a thicker wrap may result in incomplete sterilization of the da Vinci Xi endoscope